



TESTING PROCEDURE

DO NOT place anything in mouth for 15 minutes prior to taking test.

1. Open the foil package and remove the test. Observe the reactive pad on the end of the test strip. The pad should be a white or cream color. A test strip with a reactive pad that has a colored line or is otherwise discolored must be discarded.
2. Saturate the reactive pad with saliva and immediately start timer.
3. At four (4) minutes, observe the results on the reactive pad. The development of a distinct colored line across the reactive pad indicates the presence of alcohol greater than 0.02% B.A.C.

INTERPRETATION



Positive Result Greater Than 0.02% B.A.C.



Negative Result

Any decision based on the results of this test will be the sole responsibility of the user.