

NOBLE™

REF 55001

INSTRUCTIONS:

Do not place anything in mouth for 15 minutes prior to taking test.

- 1** Saturate pad at end of stick with saliva (from mouth or cup).
- 2** After complete saturation of pad, remove stick from mouth and read at the end of 2 minutes.
- 3** Compare reactive pad color blocks below.

INTERPRETATION:

NEGATIVE - *No color change on reactive pad*

POSITIVE - *Reactive pad changes color in 2 minutes.*

This test shall not be used to determine one's ability to legally operate a motor vehicle or other heavy equipment. Any decisions based on the results of this test will be the sole responsibility of the user.

When detecting alcohol in other fluids, the color chart below is not applicable. For further information about testing other fluids, please contact the manufacturer.

Approximate Blood Alcohol Level



0.02%

0.04%

0.08%

0.30%

0,2 ‰

0,4 ‰

0,8 ‰

3,0 ‰